

Expect More.


code[®]

MCKESSON PARAGON Integration Guide

CR1100 & CR1500

- Not Compatible with Other Models -

VERSION 02 – RELEASE DATE JUNE 2019

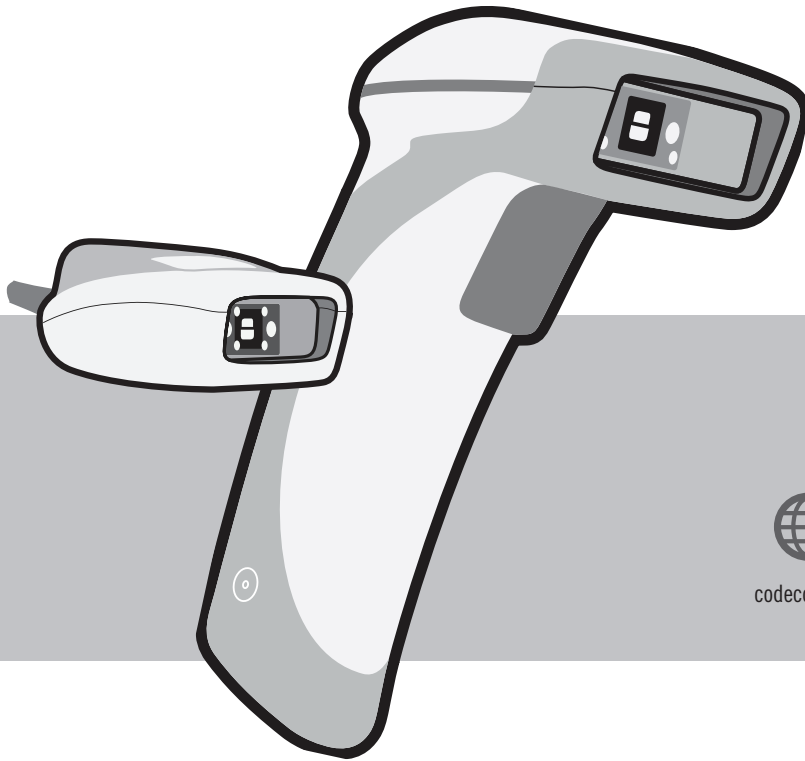

codecorp.com



Configuration Guide



[codecorporation](https://www.youtube.com/channel/UC0111111111111111111111111111111111111111)

Configuring CR1100 and CR1500 Barcode Readers

To configure a CR1100 or CR1500 to be dedicated to a patient room or workstation on wheels, scan the following barcode:



M20219_02

CR1500 Feedback Settings

Scan the preferred feedback settings (these will not work on a CR1100):



M20329_01

Beep On
Vibrate On



M20331_01

Beep On
Vibrate Off



M20330_01

Beep Off
Vibrate On

CR100 and CR1500 Reader Settings



M20111_01

Reset to factory defaults

Firmware Version Information

Scan the following barcodes to identify firmware versions:



M20361_01

Reader information